

HARTMANN




Incontinence
Management

HARTMANN Incontinence Management Catalogue

As early as in 1885, the following was certain for Paul Hartmann:
Only uncompromising quality and products based on the latest standards are the basis for future-oriented medical care.

“With our brand promise ‘Going further for health’ we have committed ourselves even more strongly to adopting the perspective of our customers and thinking ahead to offer better solutions than our competition.”

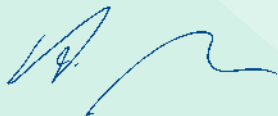Sincerely yours,


Andreas Joehle
Chief Executive Officer

Contents

Dual SkintegritY System™	2
Solutions and benefits	4
MoliCare Absorbtion	
Skin Care	
Incontinence associated dermatitis (IAD).....	6
Severe Incontinence	
MoliCare® Premium Mobile	8
Blue (6 drops)	
Purple (8 drops)	
MoliCare Premium Slip	10
Extra Plus	
Super Plus	
Fitting guide	12
Medical Skin Care	
MoliCare Skin	16
Wash Lotion	
Cleansing Foam	
Moist Skin Care Tissues	
Impregnated Wash Gloves	
Zinc Oxide Cream	
Body Lotion	
Usage guide	18
Moderate Incontinence	
MoliCare Form	22
Extra	
Extra Plus	
MoliCare Fixpants	24
Short leg	
Long leg	
Fitting guide	26
Light Incontinence	
MoliMed® Premium	30
MoliMed® for Men	32
Active	
Protect	
MoliMed® Pants Active.....	34
MoliNea® couche (maternity pad).....	35
Fitting guide	36
Personal Hygiene	
MoliNea® Underpads	40
Normal	
Plus	
Super	
Vala®.....	41
Clean	
Fit	
Protect	
Fitting and Usage guide	42


Discover the dual Skintegrity System™

An all-encompassing solution for IAD (Incontinence Associated Dermatitis), comprising of skin care and absorption products for healthy skin despite incontinence



Absorption &

- 3-Layer core
- Innovative curly fibre
- ABL (Absorption Before Leakage) tested
- Perfectly compliments the MoliCare Skin range

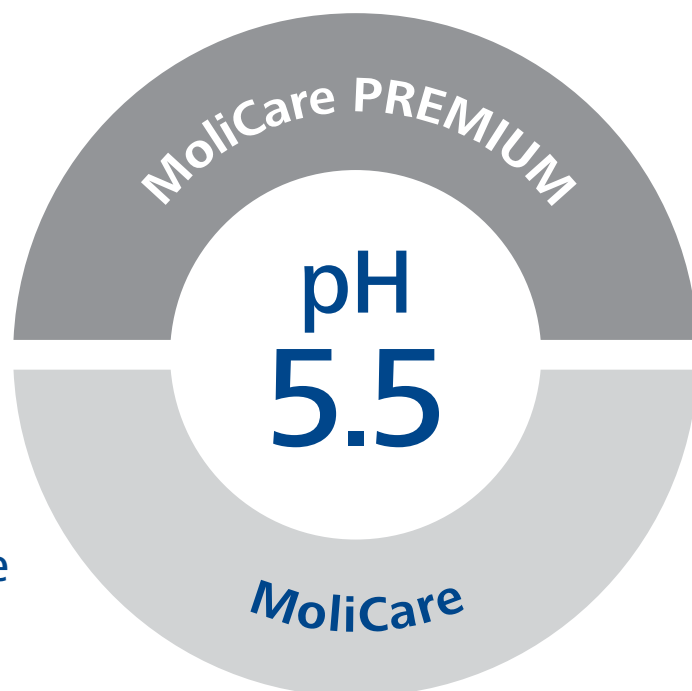



& Skin Care



- High quality
- Prevents skin irritation
- Cost and time saving
- Genuine improvements reported by customers
- Nutriskin Protection Complex
- Patented odour neutraliser

Solutions & benefits of MoliCare Absorption...

Absorption method



Irritation cause
Urine & feces



Solution
3-layer absorbent core/curly fibre



Benefit
Opportunity to reduce risk of IAD (Incontinence Associated Dermatitis)

Testing



Issue
ISO



Solution
ABL (Absorption Before Leakage)



Benefit
More accurate test results



“Effective skin protection can reduce the rate of IAD (Incontinence Associated Dermatitis) by up to 27%.”

Professor Mikel Gray, University of Virginia

Gray, M. et al. (2012) Incontinence-Associated Dermatitis: A Comprehensive Review and Update, Journal WOCN 2012; 39 (1)

... and Skin care

Cleansing



Irritation cause
Cleansing with water



Solution
Water-free application



Benefit
Reduced stress



Protection



Irritation cause
Compromised protective barrier



Solution
Nutriskin protection complex



Benefit
Time saving



Care



Irritation cause
Low quality ingredients and W/O-emulsions



Solution
W/O-emulsions




Benefit
Healthy skin



The dangers of IAD

- Incontinence associated dermatitis (IAD) is a superficial inflammation of the skin resulting from lingering urine, moisture and feces. It may be exacerbated by pressure and immobility.
- 80 percent of nursing home residents are already or will become incontinent
- 50 percent of these residents may suffer from IAD

IAD is not only detrimental to the sufferer's quality of life but it can also become expensive and time-consuming to treat.

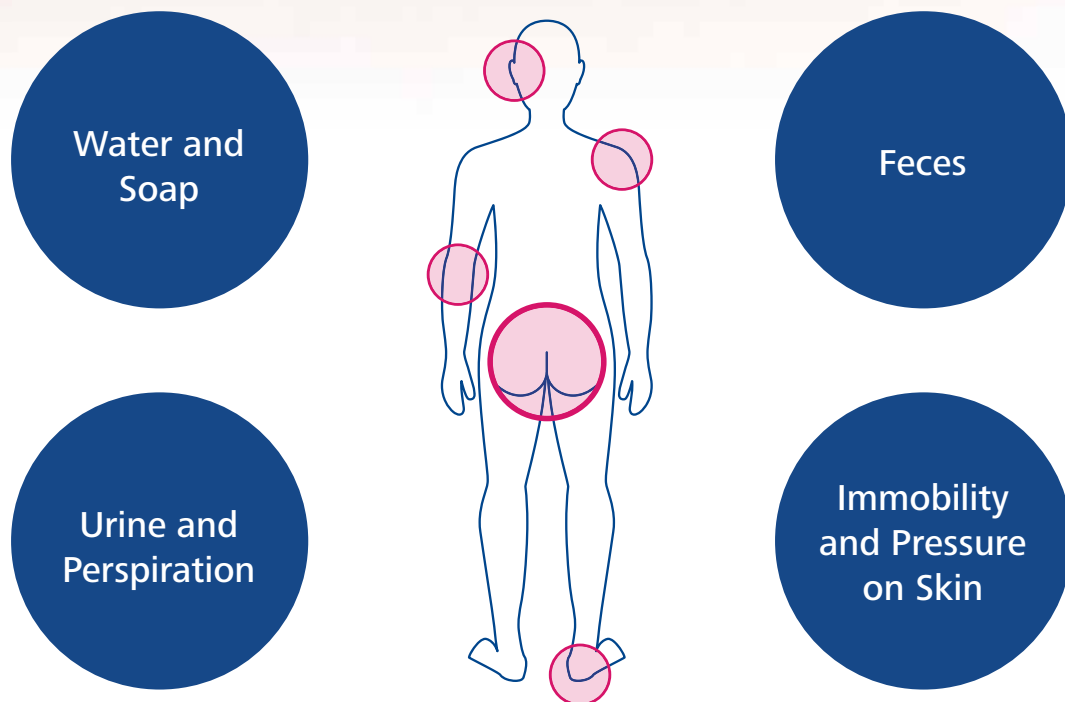

pH 5.5 skin-neutral cleansing, moisturising and protection in combination with compatible absorption products will help skin become healthy and pH balanced, for the greatest protection and prevention of IAD.




Severe Incontinence Products

MoliCare® Premium Mobile

Freedom Inside and Out

Maximum protection from
leakage even when outdoors

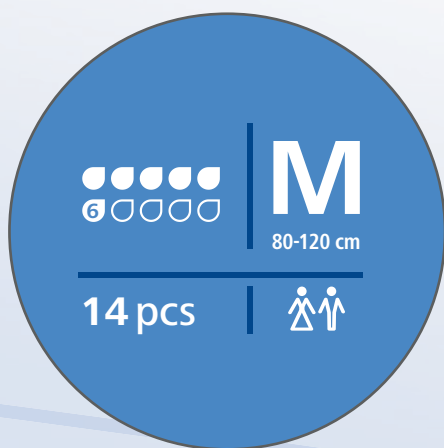

Important information
at a glance.



NEW droplet system in line with international standards.



MORE levels of absorption to fit individual needs.



NEW gender icons for faster differentiation.



Blue




Purple




Active skin protection

- Air active
- 3-layer absorbent core
- Quick-dry system




Anti leakage system

Odour neutraliser

MoliCare Premium Slip

Confidence and Security

Enabling people with incontinence
to live life with dignity

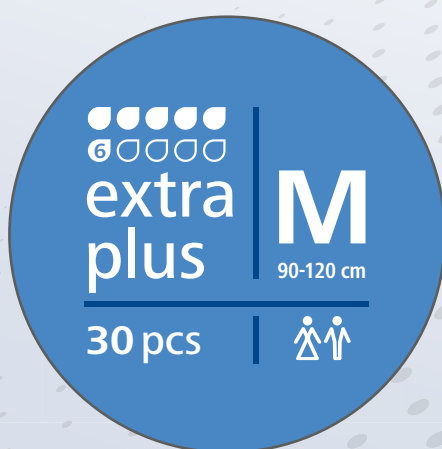

Important information
at a glance.



NEW droplet system in line with international standards.



MORE levels of absorption to fit individual needs.



NEW easy-to-understand nomenclature.



NEW gender icons for faster differentiation.



extra plus



super plus



Re-closable tapes

Active skin protection

Air active

3-layer absorbent core

Quick-dry system




Anti leakage system

Odour neutraliser

Fitting guide


MoliCare® Mobile

1

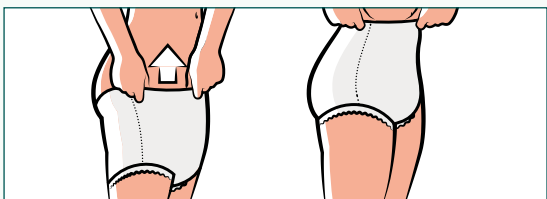

Fit the pull up pant in place of your usual underwear

2

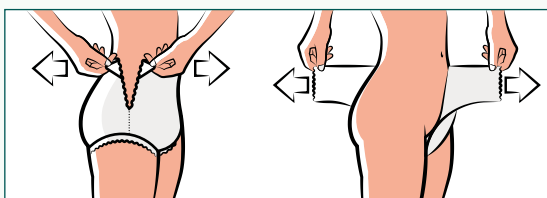

Remove by tearing the pull up pant at either side


MoliCare® Premium Slip

1




Measure the waist and hips – use the larger size.
Choose correct product size – if between sizes try the smaller size first.
Open up the brief lengthways, leaving the non-woven side panels in folded position.
Hold brief at each end and gently fold it lengthways once. This activates the inner cuffs and creates an anatomical 'cupped' shape for fitting.

2

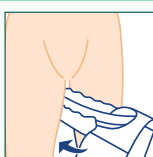

Place the 'cupped' brief between the thighs from the front – with the absorbent and completely white surface next to the skin, the absorbent area with tabs to the rear.
Unfold rear absorbent area and fit snugly to the body.
Wetness indicator should be in line with the spine.
At the front, centre the absorbent cushion in the crotch area.

3

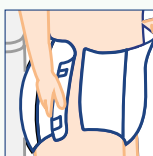

Unfold front area and fit into position, as high as possible in the groin area.
Unfold front side panels, fit snugly to the body and unfold side panels at the back.
To fasten the brief, position the upper hook and loop tabs at a slightly downwards angle onto the non-woven side panel or textile backsheet.
The lower hook and loop tabs should point diagonally upwards.

4




Check that the brief is comfortably and closely fitted.
When the stripes on the product are aligned, the super absorbent core is correctly positioned.
To open and reclose the tabs, open the blue stripe and reclose onto the textile backsheet or non-woven side panel. You can open and close the hook and loop tabs again and again.




Experience how to apply
MoliCare Premium Slip.




Product	Size	Package Contents	Article No.
MoliCare® Mobile 	XS	14 pcs	915 840
	S	14 pcs	915 831
	M	14 pcs	915 832
	L	14 pcs	915 833
	XL	14 pcs	915 834




Product	Size	Package Contents	Article No.
MoliCare® Mobile 	S	14 pcs	915 871
	M	14 pcs	915 872
	L	14 pcs	915 873
	XL	14 pcs	915 874




Product	Size	Package Contents	Article No.
MoliCare Premium Slip  extra plus	XS	30 pcs	169 248
	S	30 pcs	169 448
	M	30 pcs	169 648
	L	30 pcs	169 848
	XL	14 pcs	169 948
	S	10 pcs	169 498
	M	10 pcs	169 098
	L	10 pcs	169 198



Product	Size	Package Contents	Article No.
MoliCare Premium Slip  super plus	S	30 pcs	169 450
	M	30 pcs	169 650
	L	30 pcs	169 850
	XL	14 pcs	169 950
	S	10 pcs	169 698
	M	10 pcs	169 298
	L	10 pcs	169 398

MoliCare® Briefs with Curly Fiber

Recent study finds briefs with Curly Fiber have a potential to prevent, reduce severity, and promote healing of Incontinence Associated Dermatitis (IAD).¹

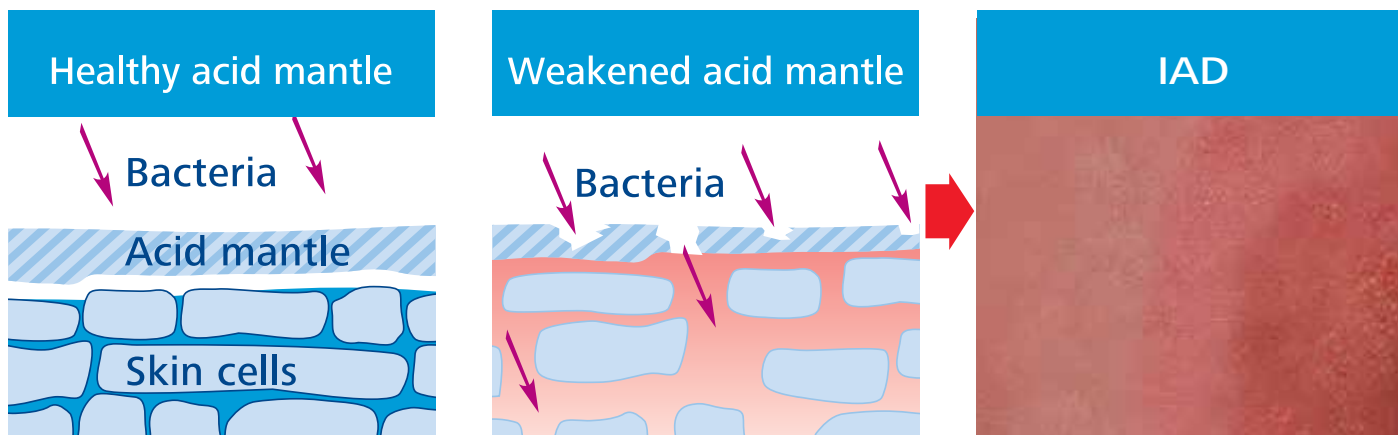

- **Healthy skin** has a slightly acidic value of **pH 4.5 to 5.5**.
- The acid mantle is an acidic film on the surface of the skin that **acts as a barrier** to bacteria, fungi, and other infections.

- Skin pH naturally **increases with age** (up to 6.0).²
- Urine, fecal matter, and perspiration may **increase skin pH up to pH 8.0**.
- Elevated pH is associated with a **breakdown** of the acid mantle, allowing bacteria to penetrate skin cells and **cause IAD**.
- Maceration and friction result in **excoriation, infection, and other complications**.

- **IAD** is an inflammation of the skin due to chronic incontinence and may be accompanied by **redness, blistering, erosion and secondary infection**.
- IAD accompanied by unrelieve pressure results in a **5-fold increase in pressure ulcer risks**.³

Conclusions

MoliCare® Briefs with Curly Fiber significantly lower/acidify pH of skin exposed to an alkaline solution with pH similar to urine/feces while standard briefs do not.¹

MoliCare® Briefs with Curly Fiber have potential to prevent IAD, reduce IAD severity, and promote IAD healing in NH residents.¹

¹ Bliss et al., Absorbent Briefs Containing Curly Fiber Lower (Acidify) Skin pH Reducing Risk for Incontinence Associated Dermatitis in Older Nursing Home Residents; WOCN Conference Symposium 2016
* ACTIVE SKIN PROTECTION means these briefs include Curly Fiber, which instantly converts the high pH value of voids to a skin-friendly pH 5.5; hypoallergenic materials, which have an extremely low potential for causing allergic reactions; superabsorbent polymer to lock-in voids and maintain a healthy, dry environment; and cloth-like, breathable side panels for increased air-permeability and optimum skin health.
² Blaak et al., Treatment of Aged Skin with a pH 4 Skin Care Product Normalizes Increased Skin Surface pH and Improves Barrier Function: Results of a Pilot Study; Journal of Cosmetics, Dermatological Sciences and Applications, 2011, 1, 50 - 58
³ Melter, Pressure Ulcer Assessment and Treatment; Wild Iris Medical Education 2014; 17



Medical Skin Care Products



MoliCare Skin

Intensive care for sensitive elderly skin

The complete solution for skin cleansing, care and protection



Moist Skin Care Tissues

- quick, practical cleansing of the genital area
- no water required, alcohol-free
- extra large size (200 x 300 mm)
- extra soft wipes
- reclosable package with lid closure
- 50 tissues per bag

Cleansing Foam

- quick and mild cleansing of heavily soiled skin areas
- no water required
- with creatine, supports the skin's own protection mechanism
- refreshing scent with odour neutralizer
- 400 ml can



Body Lotion

- intensive care for elderly skin irritated due to incontinence and bed confinement
- contains the Nutriskin Protection Complex for strengthening the skin's barrier and regeneration function
- with natural almond oil
- water-in-oil emulsion
- with integrated dispenser pump for one-hand application
- refreshing scent with odour neutralizer
- 500 ml bottle



CLEAN

p
5.

CARE



Impregnated Wash Gloves

- complete solution for whole body washing in bed
- no water required, alcohol-free
- easy and hygienic handling since hand is fully covered
- refreshing scent and high quality ingredients such as glycerin, panthenol and aloe vera
- contains 8 gloves for 8 different parts of the body



Wash Lotion

- mild cleansing for elderly skin that is irritated due to incontinence and bed confinement
- with nourishing panthenol and almond oil
- with integrated dispenser-pump for one-hand application
- refreshing scent with odour neutralizer
- 500 ml bottle

Zinc Oxide Cream

- protects irritated skin in the genital area from harmful substances
- forms a protective layer
- contains the Nutriskin Protection Complex for strengthening the skin's barrier and regeneration function
- refreshing scent with odour neutralizer
- water-in-oil emulsion
- with zinc oxide
- 200 ml tube



NSING

H
5

PROTECTION



Indication: elderly skin, stressed skin, mature and demanding (sensitive) skin
Target: mild and caring cleansing
Proceeding:
Time: as needed by the resident
How: suitable for showering and washing, for the entire body – and partial washing
Frequency: varies according to the need of the resident
Volume: recommended: at least 3 pumps /doses per wash basin or as needed for showering

Indication: fecal incontinence
Target: gentle removal of feces and encrustations
Additional benefit: odour control
Proceeding:
Time: after defecation
How: shake before use, hold bottle upside down and spray foam directly on the skin, keep distance of about 15 cm, allow to soak for about 30 sec.
Frequency: depending on the individual need
Volume: individual amount, encrustation must be completely covered Suitable for frequent use



Indication: for the care of the genital area in general and for quick cleansing on occasion, for urinal and fecal incontinence
Target: time-saving cleansing in between times (e.g. of hands), for genital area as well
Proceeding:
Time: pad change and for minor soilage (e.g. hands)
How: in genital area, always from front to back (follow hygiene guidelines to avoid spreading germs)
Frequency: as needed by resident
Volume: 1 towelette per body part

Indication: immobile residents with weakened immune system; terminal care
Target: stress-reduced cleansing – with warmed-up gloves stimulating, refreshing cleansing – with cooled gloves less physical stress for resident and caregiver
Proceeding:
Time: according to the need of the resident
How: as per figures on pack one glove per area of body
Frequency: as needed by the resident
Volume: 1 package for one bath in bed



Indication: as protective barrier in genital area, especially in cases of incontinence
Target: protective barrier in genital area
Proceeding:
Time: following every genital care procedure
How: apply to cleansed, unbroken skin
Frequency: after every genital cleansing
Volume: pea- to hazelnut-size, depending on the area

Indication: elderly skin, stressed skin, mature sensitive skin
Target: well-groomed, smooth skin
Proceeding:
Time: after washing
How: Pour into the hand (warm up) and spread evenly on the skin; suitable for the entire body. Apply a thin layer, there should be no visible film on the skin.
Frequency: as needed by the resident
Volume: if the body lotion is absorbed into the skin immediately after application, the amount used was right
Suitable for daily use





Product	Size	Package Contents	Article No.
Wash Lotion	500ml	1 pc	995 090



Product	Size	Package Contents	Article No.
Cleansing Foam	400ml	1 pc	995 091



Product	Size	Package Contents	Article No.
Moist Skin Care Tissues		50 pcs	995 092



Product	Size	Package Contents	Article No.
Impregnated Wash Gloves		8 pcs	995 093



Product	Size	Package Contents	Article No.
Zinc Oxide Cream	200ml	1 pc	995 094



Product	Size	Package Contents	Article No.
Body Lotion	500ml	1 pc	995 019



What MoliCare can do for you...



Skin
protection



Time
saving



Cost
saving



Higher
quality of life



Moderate Incontinence Products

MoliCare Form

Discreet and Comfortable
Experience



Dermatologically tested

Odour neutralizer

Anti leakage system



NEW droplet system in line with international standards.



NEW easy-to-understand nomenclature.



NEW gender icons for faster differentiation.



NEW optimal fixation with MoliCare Fixpants assortment.



extra



extra plus



Important information at a glance.



Two-piece-system for a better individual solution.

MoliCare Form & MoliCare Fixpants



Functional

- Maximum leakage protection based on high quality material.
- Open structure of the two-piece-system allows the skin to breath.

Cost saving

- Economically friendly through re-usable fixpants.
- Ecologically friendly as a result of less waste.

Comfortable

- Comfortable fit through anatomical shape and soft material.
- Easy to change on both bedridden and mobile persons.

Recommended with
MoliCare Form

MoliCare Fixpants
long leg



MoliCare Fixpants
short leg




Two-piece-system



+



=

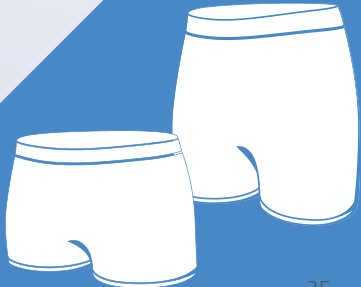

Fitting guide

Two-piece-system for a better individual solution.



Open the product and fold it in half lengthwise to activate the inner cuff system.

The smaller absorbent area of the product should be positioned at the front of the body.



Lying

Place the fixation pants with the waist elastic around the knees and turn the patient on to his/her side.

While folded, apply the product from front to back. Pull the product up in to the groin.



Unfold the product at the rear and make sure that the wetness indicator is in line with the spine.

Pull up the back of the fixation pants and turn the patient over on his/her back.



Unfold the product at the front and ensure that the leakage guards are facing up.

Pull up the fixation pants.



Make sure that product and fixation pants fit snugly while smoothing out all wrinkles and folds.



Standing

Place the fixation pants with the waist elastic around the knees.

While folded, apply the product from front to back. Pull the product up in to the groin.



Unfold the product at the rear and make sure that the wetness indicator is in line with the spine.



Pull up the back of the fixation pants.



Unfold the product at the front and ensure that the leakage guards are facing up.


Pull up the front of the fixation pants.

Make sure that product and fixation pants fit snugly while smoothing out all wrinkles and folds.




Experience
how to apply
MoliCare Form and MoliCare Fixpants



Product	Package Contents	Article No.
MoliCare Form	30 pcs	168 275
 extra		



Product	Package Contents	Article No.
MoliCare Form	30 pcs	168 375
 extra plus		




Product	Size	Package Contents	Article No.
MoliCare Fixpants (long leg)	M	50 pcs	947 741
		5 pcs	947 763
	L	50 pcs	947 742
		5 pcs	947 764
	XL	50 pcs	947 743
		5 pcs	947 765
	XXL	50 pcs	947 744
		5 pcs	947 766



Product	Size	Package Contents	Article No.
MoliCare Fixpants (short leg)	M	100 pcs	947 745
	L	100 pcs	947 746
		3 pcs	947 748
	XL	100 pcs	947 747
		3 pcs	947 750

Why skin protection is important

When it comes to incontinence in late adulthood, skin which is already less resistant is then additionally exposed to the influences of ammonia from urine and from digestive enzymes. As a result, these elements cause the pH of the skin to become alkaline. This effect is heightened when the skin is cleaned with soap and water after each episode of incontinence, because doing so weakens the skin's natural acid mantle. This then enables pathogens to penetrate the skin more easily and causing painful inflammation (dermatitis). That is why we also speak of incontinence-associated dermatitis (IAD). In order to avoid this, special attention should be paid to skin protection for those adults affected by incontinence. The Active Skin Protection concept from HARTMANN helps keep skin healthy.

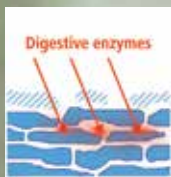

“The longer the skin is in contact with urine or stool, the greater the risk of IAD” (Incontinence Associated Dermatitis)

(Bliss D.Z.)

Bliss D. Z., Zehrer C., Savik K., Smith G., Hedblom E.; An economic evaluation of four skin damage prevention regimes in nursing home residents with Incontinence; in: Wound, Ostomy and Continence Nurses Society, March/April 2007.



Light Incontinence Products

MoliMed[®] Premium

Your professional partner

Providing ease to people with
slight bladder weakness



3-layer absorbent core
for immediate dryness
in crotch area



Odour Neutralizer to
lock odour safely away



Soft contoured shape elastic
crotch area and breathable
textile-like backsheet for
perfect wearing comfort



Skin friendly pH value 5,5
helps to stabilize the skin's
acid protection mantle



💧💧💧 mini



💧💧💧 midi



💧💧💧 maxi



Recommended to use with MoliCare Fixpants



MoliMed[®] for men

Providing two solutions
for men

Specially designed for men's individual
lifestyle needs

MoliMed[®] for men pads match
the male body perfectly with the
highest level of security, absolute
discretion, and best possible
wearing comfort.



active




The discrete protection for occasional bladder weakness

Soft breathable materials

With its textile-like feel and breathability
the backsheet provides a sense of normal
underwear and a pleasant skin climate.

Specially shaped for men

Shaped as a penis pouch. MoliMed for
men active works where liquid naturally
appears.

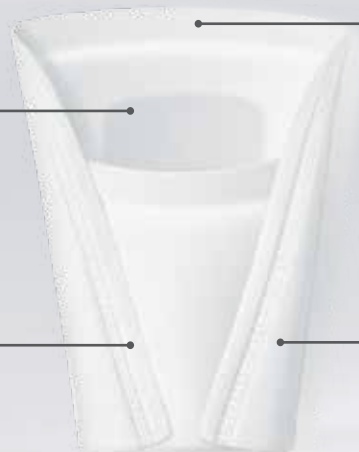

All round Protection System

The 3-layer absorbent core guides fluids
quickly into its super-absorbent core
leaving nothing but perfect dryness on
the skin.

Perfect odour neutralization

Special superabsorbent molecules lock
urine and odour safely away.



active 



protect 



Recommended to use with MoliCare Fixpants



MoliMed® for men include an adhesive strip to ensure they remain securely and discreetly in place

protect 


The high performance solution for light bladder weakness

All round Protection System

The new anti leak cuff system and the special construction of the super core guarantee high leakage protection and discretion.

Maximum discretion

Odour-binding molecules lock away odour safely and reliably.

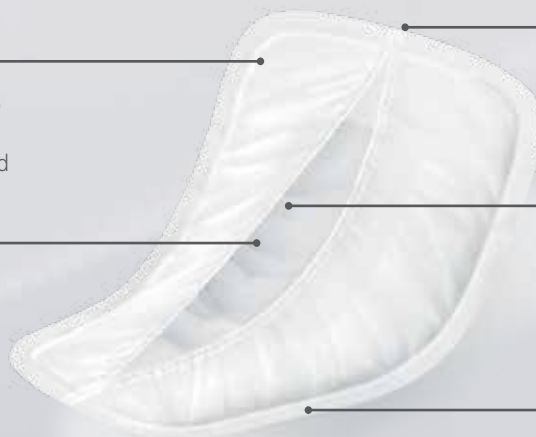

V shape

The new body shape provides a perfect fit and comfort in wearing - especially designed for men. Smaller in the crotch area, wide in the front.

Extra strong absorbency

The special structure of the topographic absorbent core ensures that even huge quantities of moisture are quickly and safely locked away – no matter where it surges.

Breathable materials

The special textile outer layer provides a comfortable skin climate and wearing comfort.

MoliMed[®] pants active

Best fit - No slip

The bladder weakness solution
for unlimited activity



Extra security

The 3-layer absorbent core guides fluids into its centre where they are stored safely with no risk of leakage.



Maximum discretion

Incorporated with a special odour neutralizer, they're not bulky, and they don't rustle.



Easy to dispose of

After use, the pants can be ripped open at the sides and disposed of safely and discreetly using the enclosed hygiene bags.

Unique shape

The new improved shape ensures that MoliMed[®] pants active fit perfectly, without slippage or wrinkling, and does not rustle.



Particular skin-friendliness

The upper layer has a highly antibacterial and "ph-buffering" effect that helps to protect and balance the skin's acidic protection mantle.



More comfort


Thanks to the new soft and breathable textile-like materials, a healthy skin condition is guaranteed.



MoliNea[®] Couche pads

Comfortable and secure
rectangular pad

Protection in the event of
childbirth or gynaecological surgery
and faecal smearing

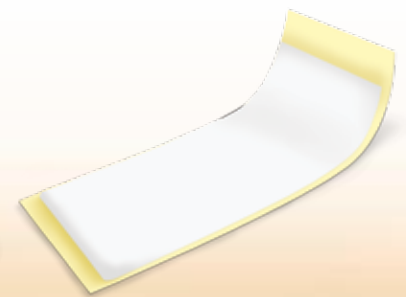

Discreet

Designed to be worn inside normal
underwear or MoliCare Fixpants

Reliable protection

Super absorbent core of cellulose fluff for better absorption
Waterproof backing sheet prevents leaks through the underwear

Security

Adhesive strip securing the pad in place

Recommended to use
with MoliCare Fixpants



Fitting guide


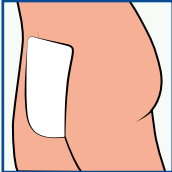

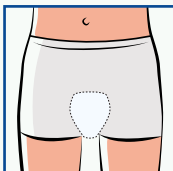
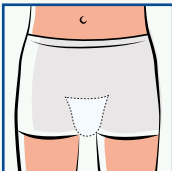
MoliMed® Premium

Suitable for slight urinary incontinence, as protection following vaginal surgery or as a comfortable, leakproof maternity pad. Can be inserted securely and discretely in the underwear or MoliCare Fixpants by an adhesive strip.

	Apply the product as closely to the body as possible		Wear within own underwear - the adhesive strip keeps the product securely in place		Soft breathable backsheet ensure maximum comfort
---	--	---	--	---	--



MoliMed® for men (protect & active)

For the management of slight urinary incontinence in men, e.g. after prostate surgery: especially suitable for mobile and active men.

			Fit the pull up pant in place of your usual underwear	Protect 	Active 
--	--	--	---	--	---


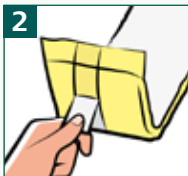

MoliMed® pants active

For women with light to moderate forms of bladder weakness, and active women who temporarily experience involuntary loss of urine e.g. after childbirth who want invisible under clothing with more security and wearing comfort than pads can offer.

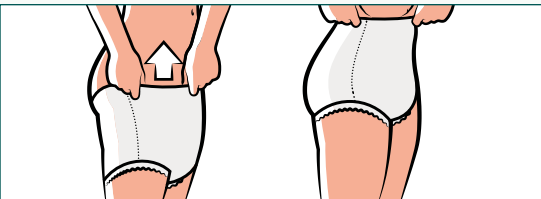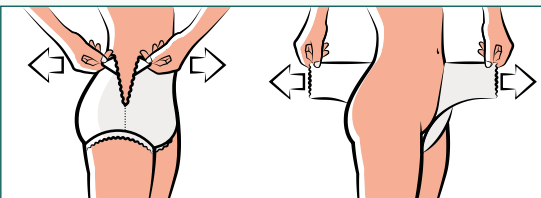1 	Fit the pull up pant in place of your usual underwear
2 	Remove by tearing the pull up pant at either side

MoliNea® couche

Protection in the event of childbirth or gynaecological surgery. Ideal solution for faecal smearing.

1 	Use MoliNea® Couche in conjunction with close fitting underwear	2 	Remove the paper from the back of the product to reveal the adhesive slip	3 	Stick the adhesive strip into the underwear to secure in place
---	--	---	---	---	--



Product	Size	Package Contents	Article No.
MoliMed® Premium	Mini	14 pcs	168 634
	Midi	14 pcs	168 644
	Maxi	14 pcs	168 654




Product		Package Contents	Article No.
MoliMed® for Men	Protect	14 pcs	168 705
	Active	14 pcs	168 600



Product	Size	Package Contents	Article No.
MoliMed® Pants Active	M	12 pcs	168 661
	L	10 pcs	168 662



Product	Size	Package Contents	Article No.
MoliNea® Couche	11 x 30 cm	56 pcs	148 510



Caregivers are starting to focus more on daily inspection of skin and skin folds on their patients to minimize the risk of *intertrigo* (superficial skin disorder) due to urinary and/or fecal incontinence.

Moreover, skin-cleansing without soap or cleansing with pH-neutral soap is highly recommended, as well as using disposable materials to prevent + treat intertrigo.

Caregivers are starting to pay more attention to the prevention of intertrigo (superficial skin disorder)



Personal Hygiene Products

MoliNea[®] underpads

Best fit - No slip

Added layer of protection
for your loved ones



normal



plus



super



Stay-dry, non woven top layer

To provide comfort during leakage and prevent skin and injury getting soaked in urine

Waterproof backing

Ensures that urine absorbed is contained within the pad

Optimized absorbent core

Concentrates the absorption capacity where it is more needed

Multifunctional

Can be used as disposable direct incontinence underpad or as additional bed protection

Variety for different types of patients

Offered in a wide range of types and sizes and thus avoid over provision

Efficient

Helps reduce laundry costs and nurses' time spent on certain tasks

Vala®

Cleanses and protects

Ensures professional body care and minimises the spread of infection

Best products for Home Care




CLEANSING

Recommended to use with:
MoliCare Skin Cleansing
foam and Wash lotion

FEEDING

SURFACE PROTECTION

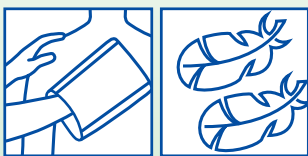


soft

- Disposable wash gloves of extra fluffy, tear-resistant Molton non-woven fabric Extra soft and absorbent
- Round handed opening for easy application
- New: ultrasonic welding
- For full body hygiene of mature skin



eco

- Disposable wipes of embossed recycled tissue
- 3-ply
- Practical use due to special folding
- For intimate incontinence cleaning

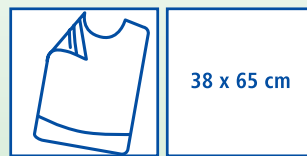

tape

- Disposable adhesive tissue bibs with plastic layer
- Kangaroo pocket
- Improved adhesive strips for securer fit
- Rounded neckline for greater comfort
- Particularly suited for protection of the immobilised person




basic

- Disposable protective sheets of 1-ply tissue with plastic underside
- Rapidly absorbs moisture and protects against wetness

Fitting and Usage guide

MoliNea® Underpads fitting guide

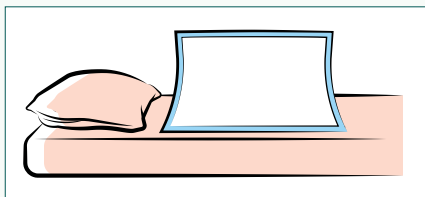


Place on bed or other surface coloured side down.

Vala® usage guide

Vala®Clean soft - Extra soft, absorbent and tear resistant disposable wash gloves

Vala®Clean eco - Soft and absorbent disposable wipe

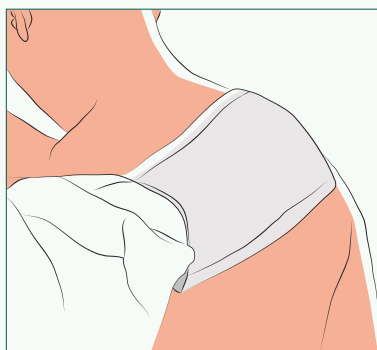


For full body hygiene with MoliCare Skin Cleansing Foam or MoliCare Skin Professional Washlotion



For intimate incontinence cleaning with MoliCare Skin Cleansing Foam or MoliCare Skin Professional Washlotion

Vala®Fit tape - Disposable tissue bibs with adhesive rounded neckline for fast, comfortable secure fit and a kangaroo pocket

Vala®Protect basic - Disposable protective bed sheets with light absorbency up and a plastic side down

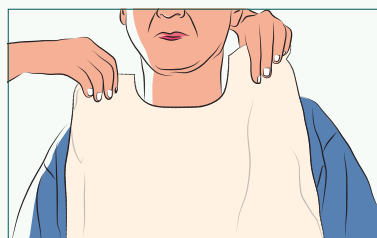

To keep the patient clothing and bed clean during meal times



To protect the surface or bed



Product	Size	Package Contents	Article No.
MoliNea® normal	40 x 60cm	30 pcs	161 220
	60 x 60cm	30 pcs	161 320
	60 x 90cm	30 pcs	161 520



Product	Size	Package Contents	Article No.
MoliNea® plus	40 x 60cm	150 pcs	161 100
	60 x 60cm	100 pcs	161 300
	60 x 90cm	100 pcs	161 600
	90 x 180cm	20 pcs	161 804



Product	Size	Package Contents	Article No.
MoliNea® super	60 x 90cm	25 pcs	161 194
	40 x 60cm	25 pcs	161 200
	40 x 60cm	100 pcs	163 100
	60 x 60cm	100 pcs	163 300
	60 x 90cm	50 pcs	163 600



Product	Size	Package Contents	Article No.
Vala® Clean Soft Washing Gloves		50 pcs	992 242



Product	Size	Package Contents	Article No.
Vala® Clean Eco Disposable Wipes	35 x 40cm	50 pcs	992 339



Product	Size	Package Contents	Article No.
Vala® Fit Tape	38 x 65cm	100 pcs	992 253



Product	Size	Package Contents	Article No.
Vala® Protect Basic	80 x 175cm	25 pcs	992 221
	80 x 175cm	100 pcs	992 228





PAUL HARTMANN Middle East
Dubai Airport Freezone
P.O. Box 54525 Dubai
United Arab Emirates
Phone: +971 4 2996996
Fax: +971 4 2996998

me.hartmann.info



**Going further
for health**